

Ambu® aScope™ 4 Broncho Regular 5.0/2.2

Flexible Endoscope - Single Use



Key Benefits

- Instant availability
- No risk of cross-contamination
- Cost effective
- High degree of usability
- Clear, sharp images
- Smooth and easy navigation

Ambu® aScope™ 4 Broncho Regular 5.0/2.2

Guaranteed sterility, ready when you are.

The single-use concept of aScope 4 Broncho once and for all solves the problem of accessibility, simplifies set-up for procedures, eliminates the need for complex cleaning procedures and minimizes the risk of cross-contamination. It comes sterile straight from the pack, offering improved patient safety and workflow.

The aScope 4 Broncho Regular offers high quality bronchoscopy for a broad range of procedures. When connected to one of the portable monitors, Ambu aView™ 2 Advance or Ambu aView™, aScope 4 Broncho Regular gives you clear visibility of the near and distal bronchial segments. The clear, sharp images and adaptive light control make identification of lung anatomy and navigation in the bronchial tree easy.

The high bending angles and precise tip motion of aScope 4 Broncho Regular allows you to easily navigate into all segments of the lung. The high quality handle is designed with a shaped, rippled surface to provide a comfortable grip throughout the procedure. The lightweight and ergonomic design makes aScope 4 Broncho Regular easy to use.

The aScope 4 Broncho Regular has a working channel that is ideal for a wide range of bronchoscopy procedures. This ranges from intubation and training of intubation skills to bedside bronchoscopy including BAL & PDT procedures.

Specifications

Optical system	
Field of View	85°
Direction of View	0° (forward viewing)
Depth of Field	6-50 mm
Illumination method	LED
Insertion portion	
Bending section	180° up, 180° down
Insertion cord diameter	5.0 mm (0.20")
Distal tip diameter	5.4 mm (0.21")
Maximum diameter of insertion portion	5.5 mm (0.22")
Minimum endotracheal tube size (inner diameter)	6.0 mm
Minimum double lumen tube size (inner diameter)	41 Fr
Working length	600 mm (23.6")
Channel	
Average inner diameter	2.2 mm (0.087")
Minimum instrument channel width	2.0 mm (0.079")
Suction connector	
Connecting tube inner diameter range	6.5 mm - 9.5 mm
Operating environment, storage & transportation	
Temperature - transportation	10 ~ 40° C (50 to 104° F)
Temperature - operating environment	10 ~ 40° C (50 to 104° F)
Temperature - storage	10 ~ 25° C (50 to 77° F)
Relative humidity	30 ~ 85%
Atmospheric pressure	80 ~ 109 kPa
Sterilisation	
Method of sterilisation	ETO



Ambu aScope 4 Broncho Regular and Ambu aView 2 Advance




The ergonomic and lightweight handle is designed to give you a firm and comfortable grip

Ordering specifications

Item no.	Quantity (Ambu® aScope™ 4 Broncho Regular 5.0/2.2)
477001000	5

Introducer

To facilitate introduction of Luer Lock syringes and soft endoscopic accessories through working channel

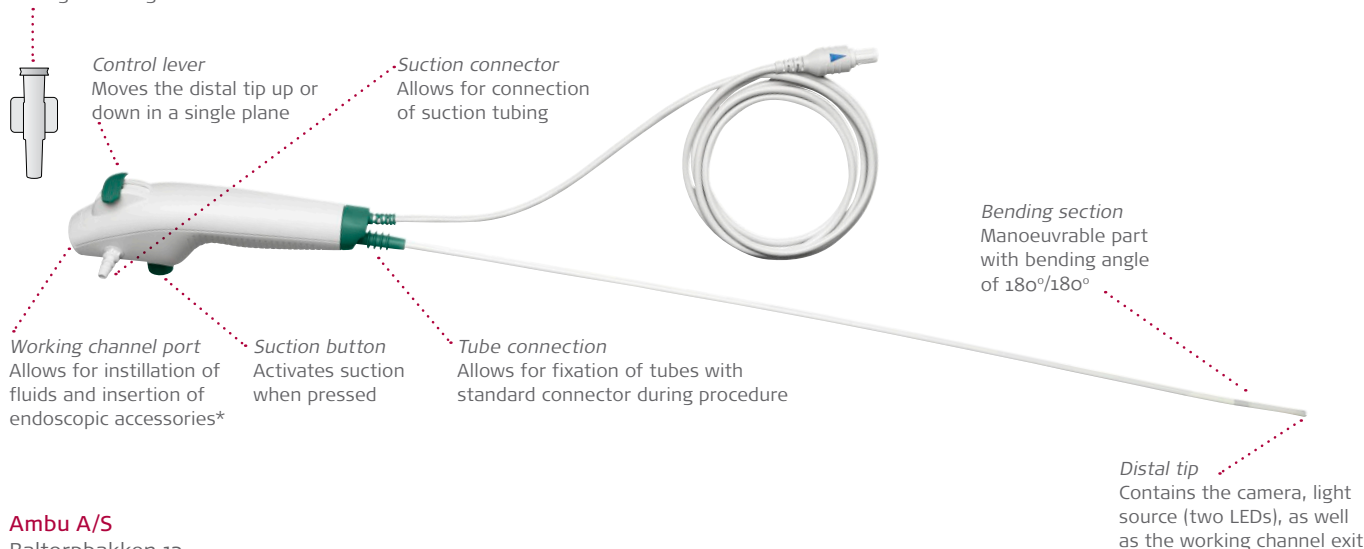

Ambu A/S

Baltorpbakken 13
DK - 2750 Ballerup
Denmark
Tel. +45 7225 2000
Fax +45 7225 2053
www.ambu.com

*There is no guarantee that accessories selected solely using this minimum instrument channel width will be compatible in combination.