

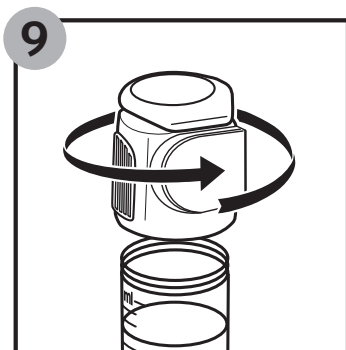
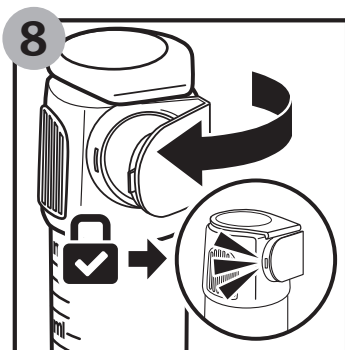
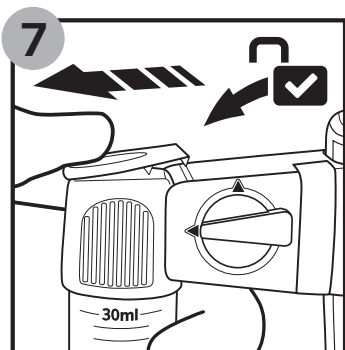
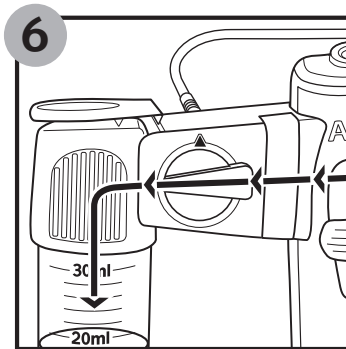
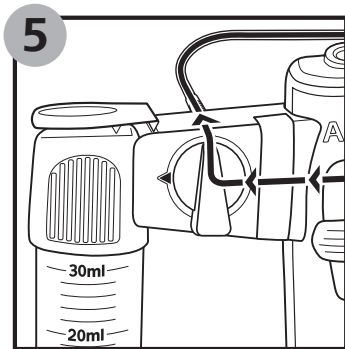
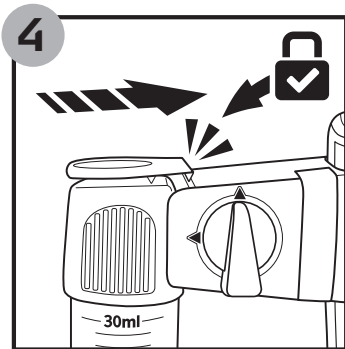
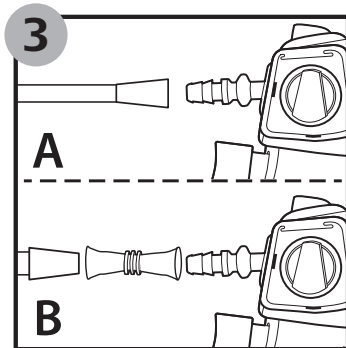
# Instruction for Use

## Ambu<sup>®</sup> aScope BronchoSampler™

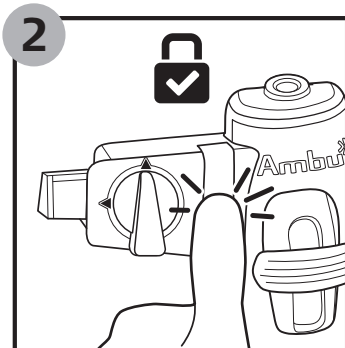For use with Ambu<sup>®</sup> aScope™ 4 Broncho.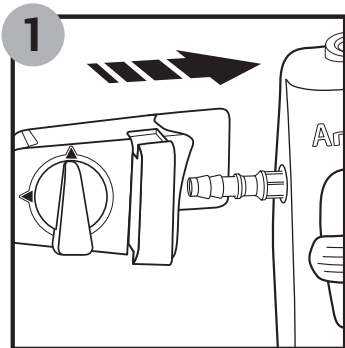

CE

IVD Pat. Pending

Ambu is a registered trademark and aScope BronchoSampler is trademark of Ambu A/S.

<b>Contents</b>	<b>Page</b>
English (Instruction for Use) .....	04-09
Български (Инструкция за употреба).....	10-15
Čeština (návod k použití) .....	16-21
Dansk (Brugsanvisning).....	22-27
Deutsch (Bedienungsanleitung) .....	28-33
Ελληνικά (Οδηγίες χρήσης) .....	34-39
Español (Instrucciones de uso).....	40-45
Eesti (Kasutusjuhend).....	46-51
Käyttöohje (suomi) .....	52-57
Français (Mode d'emploi).....	58-63
Hrvatski (Upute za upotrebu) .....	64-69
Magyar (Használati útmutató) .....	70-75
Inglese (Istruzioni per l'uso).....	76-81
英語 (使用説明書) .....	82-87
Lietuvių (naudojimo instrukcija) .....	88-93
Latviešu val. (Lietošanas instrukcija).....	94-99
Nederlands (Gebruiksaanwijzing).....	100-105
Norsk (Bruksanvisning) .....	106-111
Angielski (Instrukcja użytkowania) .....	112-117
Português (Instruções de utilização) .....	118-123
Română (Instrucțiuni de utilizare).....	124-129
Русский (инструкция по применению) .....	130-135
Slovensky (Návod na použitie) .....	136-141
Slovenščina (Navodila za uporabo).....	142-147
Svenska (bruksanvisning).....	148-153
Türkçe (Kullanma Talimatı).....	154-159
中文 (使用说明) .....	160-165

## 1. Important Information – read before use

Read these safety instructions carefully before using the aScope BronchoSampler™. The Instruction for Use may be updated without further notice. Copies of the current version are available upon request. Please be aware that these instructions do not explain or discuss clinical procedures. They describe only the basic operation and pre-cautions related to the operation of the aScope BronchoSampler™. Before initial use of the aScope BronchoSampler™, it is essential for operators to have received sufficient training in clinical endoscopic techniques and to be familiar with the intended use, cautions and indications mentioned in these instructions.

### 1.1. Intended Use

aScope BronchoSampler™ is a sterile single-use device designed as an add-on to aScope™ 4 Broncho during a Broncho Alveolar Lavage (BAL) or Bronchial Wash (BW) procedure. aScope BronchoSampler™ is in combination with an aScope 4 Broncho device designed to enable aspiration and collection a fluid sample(s) from the bronchial or alveolar part of the lung.

### 1.2. Indications for Use

aScope BronchoSampler™ is for use in a hospital environment. aScope BronchoSampler™ is a single-use device designed for use in adults.

### 1.3. Cautions

Failure to observe these cautions may result in patient injury or damage to the equipment.

**Ambu is not responsible for any damage to the system or patient injury resulting from incorrect use.**

#### CAUTIONS

1. For single use only. Use on other patients can cause cross contamination. Do not soak, rinse, or sterilize this device as these procedures may leave harmful residues or cause malfunction of the device.
2. Only to be used by skilled professionals trained in clinical endoscopic techniques and procedures.

## 2. System Description

aScope BronchoSampler™ is for use together with aScope 4 Broncho only. For information about the use of aScope 4 Broncho, please refer to the aScope 4 Broncho Instruction for Use.

### 2.1. System Parts

#### Ambu® aScope BronchoSampler™ – single use device:

#### Part numbers:

480200000 aScope BronchoSampler™

480300000 aScope BronchoSampler™ SC

aScope BronchoSampler™ is not available in all countries. Please contact your local sales office.

Product Name	Packaging configuration	Sampler bridge	Sample container	Suction adapter
aScope BronchoSampler™		1 pc	2 pcs	1 pc
aScope BronchoSampler™ SC			1 pc	

#### Compatible devices: Ambu aScope 4 Broncho

#### Part numbers:

aScope 4 Broncho Large 5.8/2.8

478001000

aScope 4 Broncho Regular 5.0/2.2

477001000

aScope 4 Broncho Slim 3.8/1.2\*


476001000

aScope 4 Broncho Slim, aScope 4 Broncho Regular and aScope 4 Broncho Large are not available in all countries. Please contact your local sales office.

\* aScope BronchoSampler™ has limited sampling and suction performance on aScope 4 Broncho Slim.

## 2.2. aScope BronchoSampler™ parts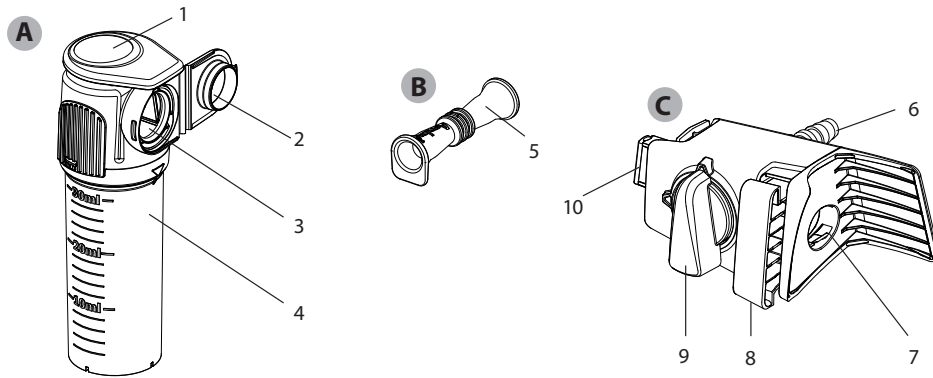

Item	Function
<b>A</b> Sample container	Connects to the sampler bridge and holds the retrieved sample
<b>B</b> Suction adapter	For connection to a male suction adapter and the suction connector
<b>C</b> Sampler bridge	Connects to the aScope 4 Broncho, and creates a closed loop for sampling and/or suctioning

No.	Part	Function	Material
1	Flip top	Acts as click-on locking mechanism for sample container. To be pressed down anteriorly to disengage locking mechanism	PP
2	Sample container lid	For closing and protecting the sample	PP
3	Connection interface	For connection of sampler bridge and sample container	Silicone PP
4	Sample container glass	For storage of sample	HIPS

No.	Part	Function	Material
5	Suction adapter	For connection to a male suction adapter and the suction connector	TPE

No.	Part	Function	Material
6	Suction connector	For connection of suction tubing	PP
7	aScope 4 Broncho connection interface	For introduction of aScope 4 Broncho suction connector, to establish connection	MABS TPE
8	Attachment lock	Locks the sampler bridge to the aScope 4 Broncho	POM
9	Flow switch	Determines suction flow direction and switches between general suction and sampling	POM
10	Connection interface	For connection of sampler bridge and sample container	MABS Silicone

### 3. Explanation of Symbols Used

	Do not use if the product sterilisation barrier or its packaging is damaged.		This product is not made with PVC.
	Single use product, do not reuse.		This product is not made with phthalates.
	Sterile Product, sterilisation by ETO.		CE mark. The product complies with the EU Council directive concerning In-Vitro Diagnostic Medical Devices 98/79/EC.
	Consult Instruction for Use.		Manufacturer.
	Warning.		Use By, followed by YYYY-MM-DD.
	Temperature limitation: storage between 10-40 °C (50-104 °F).		Reference Number.
	This product is not made with natural rubber latex.		Lot Number.

### 4. Use of Product

Numbers in gray circles below refer to illustrations on page 2.

#### 4.1. Preparation and Inspection of aScope BronchoSampler™

##### Visual inspection

1. Check that the packaging is intact.

##### Refer to the aScope 4 Broncho Instruction for Use for inspection, preparation and use of the aScope 4 Broncho. Preparation for use of aScope BronchoSampler™ with aScope 4 Broncho

1. Carefully connect the sampler bridge with the aScope 4 Broncho, by mounting the sampler bridge on the aScope 4 Broncho suction connector. Ensure a tight fit. **1**
2. Press the attachment lock button to secure connection. **2**
3. Carefully connect the vacuum suction tubing to the suction connector according to the supplier's manual. If vacuum suction tubing has male connector, use the suction adapter to ensure fit. **3**
4. Attach the sample container to the sampler bridge by joining the connection interfaces and pressing the sample container towards the sampler bridge. Ensure the sample container is clicked on. Do not use preservatives in the sample container. **4**

#### 4.2. Operating the aScope BronchoSampler™

1. If suction is deemed relevant before sampling, ensure that the flow switch is aligned vertically and pointing upwards, and perform suction by pressing the suction button on the aScope 4 Broncho in accordance with Instruction for Use for aScope 4 Broncho. **5**
2. To prepare for sampling of specimen, turn the flow switch counter-clockwise towards the sample container, and align it horizontally. **6**
3. Instillation of fluids should be performed in accordance with Instruction for Use for aScope 4 Broncho.
4. Perform sampling by pressing the suction button on the aScope 4 Broncho. Do not fill the sample container above the maximum of the volume indicator scale.

1. If suction is deemed relevant, turn the flow switch clockwise to align it vertically and pointing upwards, and press the suction button on the aScope 4 Broncho. When suction is complete, turn the flow regulator counter-clockwise towards the sample container, and align it horizontally.
2. Remove the sample container by pressing the flip top, and pull the sample container from the sampler bridge. **7**
3. Close the sample container by pressing the sample container lid towards the sample container interface and engage the click-lock. Once the click-lock is engaged, do not re-open the lid. **8**

### 4.3. Post Procedure

1. If deemed relevant, detach the sampler bridge from the aScope 4 Broncho by disengaging the attachment lock, and pull the sampler bridge off the aScope 4 Broncho. Disengage suction by pulling the suction tube off the sampler bridge.
2. If deemed relevant, the sample can be accessed by unscrewing the sample container glass counter-clockwise. It is recommended to handle contaminated substances in accordance with local guidelines. **9**
3. Transportation and handling of sample container must be performed in accordance with local guidelines for transportation and handling of hazardous material.

#### Final steps

aScope BronchoSampler™ is a single use device. aScope BronchoSampler™ is considered infected after use and must be disposed in accordance with local guidelines for collection of infected medical devices.

## 5. Technical Product Specifications

### 5.1. Standards Applied

The aScope BronchoSampler™ function conforms with:

- ISO 8600-1: Endoscopes - Medical endoscopes and endotherapy devices - Part 1: General requirements
- EN 14254: In Vitro Diagnostic medical devices – Single-use receptacles for the collection of specimens, other than blood, from humans

### 5.2. aScope BronchoSampler™ Specifications

Suction Connector <sup>1</sup>	Specification
Connecting tube with female interface.	ID = Ø7 ± 1 mm
Connecting tube with male interface. Applicable when suction adapter is used.	OD = Ø7 ± 1 mm
1. Please check for compatibility with suction tubing before use	
Sample Container	Specification
Sample Container volume	30 ml
Storage and transportation	Specification
Storage and transportation temperature	10-40°C (50-104°F)
Operating environment	Specification
Temperature	10 - 40°C (50-104°F)
Sterilisation	Specification
Method of sterilisation	ETO

## 6. Trouble Shooting

If problems occur with the system, please use this trouble shooting guide to identify the cause and correct the error.

<b>Problem</b>	<b>Possible cause</b>	<b>Recommended action</b>
Absent or reduced suction or sampling capability.	No sample container is attached, or poor connection of sample container.	Connect or re-connect sample container to the sampler bridge.
	Sampler bridge may not be connected properly to the aScope 4 Broncho.	Re-connect the sampler bridge to aScope 4 Broncho, and press the attachment lock button to secure connection.
	Suction pump is not turned on or not connected.	Turn the pump on and check the suction line connection.
	Vacuum suction tubing is not connected properly to suction connector, or to suction adapter.	Ensure that the parts are properly connected. If suction is still absent or reduced change to another vacuum suction tubing.

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